

Binax NOW Training Competency Quiz

Return this form to MDHHS Bureau of Laboratories at anglewiczc@michigan.gov or fax 517-335-9631

Trainee Name (Please Print) _____


Training Location (Please Print) _____ Date _____

1. What is the time range that the BinaxNOW Antigen test card can be interpreted after beginning the test?
 - a. 5-10 minutes
 - b. 10-20 minutes
 - c. 15-30 minutes
 - d. 30-45 minutes
2. Draw a line to match the following test interpretations with the BinaxNOW antigen test card.



Pink/Purple Control Line

Positive Result

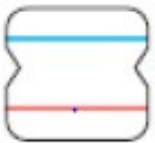

Blue Control Line

Sample Line

Invalid (Repeat Test-**DO NOT REPORT**)



Pink/Purple Control Line

Pink/Purple Sample Line

Negative Result



Sample Line Only

3. True or False (Circle correct answer) It is acceptable to use any swab to collect the sample even if it doesn't come from the BinaxNOW test kit?
4. True or False (Circle correct answer) The medications Bactroban and Centany can interfere with the BinaxNOW test?
5. True or False (Circle correct answer) It is ok to discard contaminated gloves in a regular waste container instead of a biohazard waste container?