

MONOJECT™ Safety Needle

Instruction For Use



1
Open Monoject Magellan package by peeling back paper tabs.



2
Remove protective needle sheath.



3
Open Monoject Magellan package by peeling back paper tabs.

Lock the safety shield using any of the following activation methods



4
...thumb

or



...finger

or



...flat surface



5
Verify locked position through audible and tactile "click". Locked position will completely cover needle tip.



6
Once safety shield is locked, immediately dispose of needle and syringe in an approved sharps container.



Needle Gauge Colors

18
Pink

19
Cream

20
Yellow

21
Green

22
Black

23
Blue

25
Orange

All MONOJECT Safety Needles are ISO color-coded.



COVIDIEN

positive results for life™

COVIDIEN, COVIDIEN with logo and
™ marked brands are trademarks of Covidien AG or its affiliates.
©2007 Covidien AG. All rights reserved.

H-6077-10M-0907

15 HAMPSHIRE STREET
MANSFIELD, MA
02048

1.800.962.9888
508.261.8000

www.covidien.com